

# BruxZir® **NOW**

“The Strength You Trust”



## The **only fully sintered zirconia** for in-office use

- Indicated for posterior crowns demanding maximum strength
- Mill in under 40 minutes\*
- Exclusively designed for Glidewell milling systems
- The No. 1 brand of full-contour zirconia, with over 16 million restorations prescribed worldwide



MAXIMUM  
STRENGTH



NO OVEN  
NEEDED



MINIMAL  
PREPARATION

Before



After

This patient's failing composite restoration was replaced with a BruxZir NOW crown, which was milled in under 36 minutes.

“BruxZir® NOW restorations have a natural luster straight from the mill that meets patient and dentist approval alike.”

– Paresh Patel, DDS | Mooresville, NC

For more information  
[glidewelldental.com](http://glidewelldental.com) ■ 800-854-7256



**GLIDEWELL  
DENTAL**

## BENEFITS

- No oven or post-milling processing needed; just polish and cement
- Conventionally cemented, no bonding required
- With minimal preparation requirements similar to full-cast gold, BruxZir NOW allows patients to keep more of their existing dentition



Sold in 3-block packs that include 3 single-use burs

## READY TO DELIVER IN UNDER 50 MINUTES



MILL

+



FINISH/POLISH

+



FURNACE  
(No oven required)

+



CEMENT

=



TOTAL TIME

## SPECIFICATIONS

### MATERIAL

- Pre-shaded, fully sintered zirconia
- Flexural Strength: 1,100 MPa

### INDICATIONS

- Single-unit posterior restorations


### IDEAL FOR

- Cases with limited interocclusal space
- Restorations with < 1 mm margin thickness

### SHADES

- Available in 14 shades that correspond to the VITA® classical shade system: A1, A2, A3, A3.5, A4, B1, B2, B3, B4, C1, C2, C3, D2, D3

## EASY PREPARATION

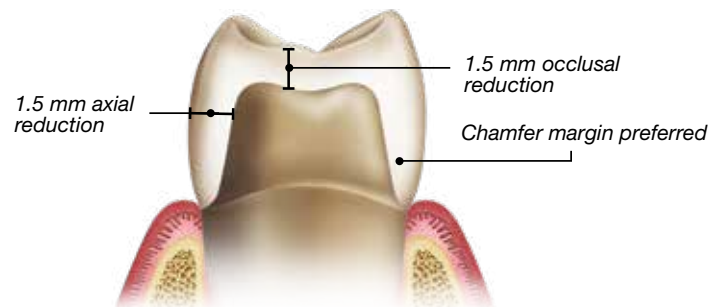

- 0.7 mm incisal/occlusal reduction required; 1.5 mm recommended
- Chamfer or shoulder margins preferred; feather edge OK
- All axial walls must be convergent

\*Milling times vary based on prescription and mill used. Average milling time shown reflects use of the fastmill.io™ In-Office Unit.

VITA is a registered trademark of VITA Zahnfabrik.

MKT 10317\_1.0 PK-268534-060618

For more information  
[glidewelldental.com](http://glidewelldental.com) ■ 800-854-7256

