



BioTemps® **NOW**

“The Temporary You Admire”



Affordable PMMA material designed for **esthetic, in-office-milled temporaries**

- Recommended for temporary full-coverage anterior and posterior crowns
- Mills in under 10 minutes*
- Available exclusively for Glidewell milling systems
- Low water sorption means long-lasting strength and shade retention



PATIENT-PLEASING
RESULTS



NO OVEN
NEEDED



RETAINS GINGIVAL
POSITION

Before



After



This patient presented with failing restorations and recurrent decay on teeth #12–15. Because of the number of units involved, BioTemps® NOW provisional restorations were milled and seated for a durable and comfortable solution while awaiting final treatment.

“

For our cosmetically-driven cases, BioTemps NOW is an excellent option to meet the demand for esthetics while the final restorations are being fabricated. Furthermore, the pre-treatment scanning with the fastdesign.io software allows the temporaries and final restorations to mimic our diagnostic models and wax-ups.”

- Joshua Prentice, DDS | Bend, OR

For more information
glidewelldental.com/milling-blocks ■ 844-949-7184



BENEFITS

- Give your patients the ability to preview their final results
- Deliver excellent abrasion resistance for up to six months of wear time



Sold in 5-block packs

READY TO DELIVER IN UNDER 15 MINUTES



SPECIFICATIONS

MATERIAL

- PMMA
- Flexural strength: ≥ 50 MPa


IDEAL FOR

- Temporary full-coverage anterior and posterior crowns

SHADES

- 6 VITA® Classical Shades, 1 Bleach Shade: A1, A2, A3, A3.5, B1, C2, BL

EASY PREPARATION

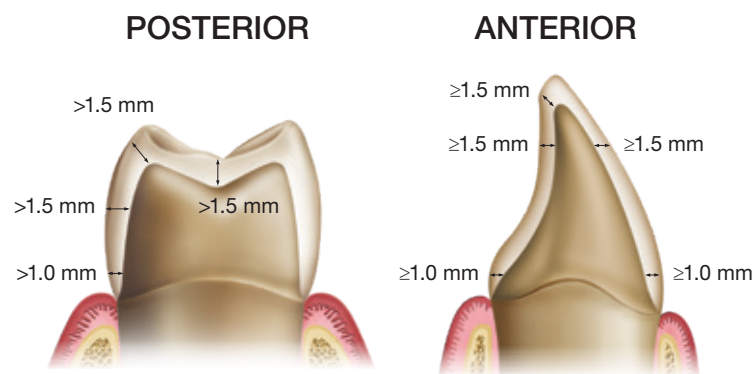

- Wall thickness should be at least 1.5 mm (1.0 mm at margins)
- Margins should be prepared with a deep chamfer or rounded shoulder

*Milling times vary based on prescription and mill used. Average milling time is shown. VITA is a registered trademark of VITA Zahnfabrik.

MKT-10465_4 PK-281915-022720

NEW!

BioTemps® **NOW**

BRIDGE BLOCK

"The Temporary You Admire"



Affordable PMMA material designed for **multi-unit temporaries milled in-office**

- Recommended for temporary full-coverage anterior and posterior 3- to 4-unit bridges
- Mills in under 30 minutes*
- Available exclusively for Glidewell milling systems
- Low water sorption means long-lasting strength and shade retention



NO OVEN
NEEDED



RETAINS GINGIVAL
POSITION



PATIENT-PLEASING
RESULTS

Before



After



This patient's failing PFM bridge on teeth #7-9 was removed to make way for individual zirconia crowns. An implant was placed for missing tooth #9, after which a BioTemps® NOW bridge was milled in-office to provisionalize the teeth and protect the site during the healing phase.

For more information
glidewell.com/milling-blocks ■ 844-949-7184



BENEFITS

- Give your patients the ability to preview their final results
- Deliver excellent abrasion resistance for up to six months of wear time

SPECIFICATIONS

MATERIAL

- PMMA
- Flexural strength: ≥ 50 MPa

IDEAL FOR

- Temporary full-coverage anterior and posterior 3- to 4-unit bridges



SHADES

- 6 VITA® Classical Shades, 1 Bleach Shade: A1, A2, A3, A3.5, B1, C2, BL

READY TO DELIVER IN UNDER 35 MINUTES



EASY PREPARATION

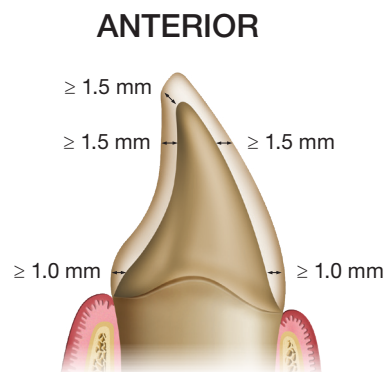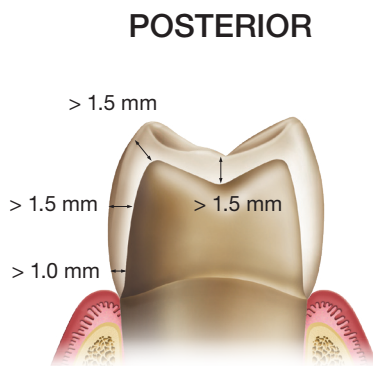

- Wall thickness should be at least 1.5 mm (1.0 mm at margins)
- Margins should be prepared with a deep chamfer or rounded shoulder

*Milling times vary based on prescription and mill used. Average milling time for 3-unit bridge is shown.
VITA is a registered trademark of VITA Zahnfabrik.

MKT-012934_1 PK-2099813-040221

For more information
glidewell.com/milling-blocks ■ 844-949-7184

