


NEW!

BruxZir[®] *esthetic* NOW

Beauty in a Single Visit

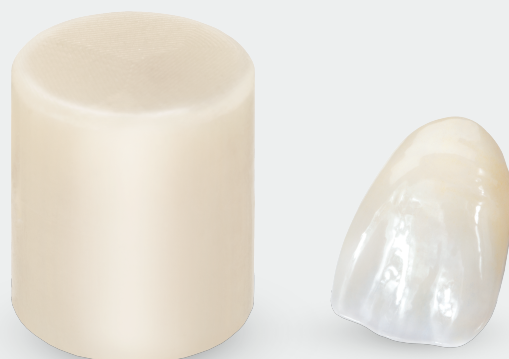

Fully sintered, highly esthetic zirconia for in-office use

- Recommended for anterior crowns demanding high strength and excellent esthetics
- Mills in under 45 minutes*
- Available exclusively for Glidewell milling systems
- Optimized combination of strength and beauty



PATIENT-PLEASING
RESULTS



NO OVEN
NEEDED



ANTERIOR
CROWNS



HIGH
STRENGTH

For more information
glidewell.com/milling-blocks ■ 844-949-7184




BENEFITS

- Lifelike translucency engineered for the anterior
- Deliver restorations right from the mill with no post-processing required
- Easy delivery — conventional cementing with no need to bond
- Preserve your patients' existing dentition with minimal preparation requirements

BruxZir[®]
esthetic **NOW**

READY TO DELIVER IN UNDER 50 MINUTES

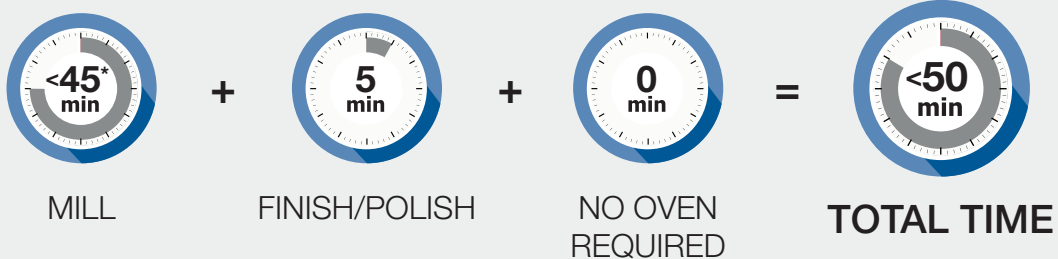

SPECIFICATIONS

MATERIAL

- Pre-shaded, fully sintered zirconia
- Flexural strength: 870 MPa


IDEAL FOR

- Single-unit anterior restorations, including bicuspid
- Cases with limited interocclusal space
- Restorations with < 1 mm margin thickness

SHADES

- 16 VITA[®] Classical Shades: A1, A2, A3, A3.5, A4, B1, B2, B3, B4, C1, C2, C3, C4, D2, D3, and D4
- 2 Bleach Shades: BL1 and BL White

EASY PREPARATION

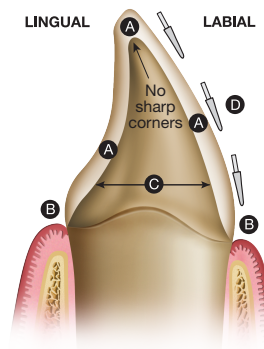

- A. 1.25 mm ideal reduction (0.7 mm minimum)
- B. Chamfer or modified shoulder margins preferred
- C. Axial walls must be convergent (avoid undercuts)
- D. Preparation should be cut in three planes
- E. To achieve optimal impression quality, gingival retraction is necessary for preparations with subgingival or equigingival margins

*Milling times vary based on prescription and mill used. Average milling time shown reflects use of the fastmill.io™ In-Office Mill.

VITA is a registered trademark of VITA Zahnfabrik.

MKT-012751_1 PK-1524218-040921

For more information
glidewell.com/milling-blocks ■ 844-949-7184

