




CAMouflage® NOW

“The Speed You Desire”



The combination of **speed**, **esthetics** and **durability** you desire

- Recommended for inlays, onlays, and anterior and posterior crowns
- Mills in under 10 minutes*
- Available exclusively for Glidewell milling systems
- Long-lasting shade and gloss retention

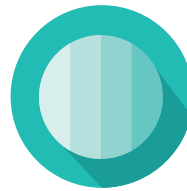

RETAINS
SHADE



RESISTS
STAINS



NO OVEN
NEEDED

Before



After

This patient presented with failing direct composite restorations on teeth #30 and #31. A CAMouflage® NOW A1 overlay on tooth #30 and MOBL onlay on tooth #31, provided a minimally invasive single-visit solution.

“CAMouflage NOW is an excellent restorative alternative for indirect dentistry. I can provide high-quality conservative restorations to my patients in a matter of minutes. This, in turn, has increased the productivity of my practice.”

– Chad Duplantis, DDS | Fort Worth, TX

For more information
glidewell.com/milling-blocks ■ 844-949-7184



BENEFITS

- Shade stability long after delivery
- Rapid patient delivery with no oven wait
- Choose from a wide range of lifelike shades that blend well with existing dentition



Sold in 5-block packs

READY TO DELIVER IN UNDER 10 MINUTES



SPECIFICATIONS

MATERIAL

- Approximately 80% ceramic-filled nanohybrid resin composite
- Flexural strength: 193 MPa

IDEAL FOR

- Single-unit anterior and posterior crowns
- Inlays and onlays

SHADES

- 7 VITA® Classical Shades, 5 in 2 translucencies: A1HT/LT, A2HT/LT, A3HT/LT, A3.5HT/LT, B1HT/LT, C2LT, D2LT
- 1 VITA Bleach Shade: BLLT

EASY PREPARATION

ONLAY



- Prepare cavity walls tapered 5–6° to the long axis.
- Wall thickness should be ≥ 1.5 mm.
- Isthmus thickness should be ≥ 1.5 mm.

*Milling times vary based on prescription and mill used. Average milling time is shown.
VITA is a registered trademark of VITA Zahnfabrik.

MKT-10487_6 PK-280353-032322