

# BruxZir<sup>®</sup> *esthetic* NOW

*"Beauty in a Single Visit"*



## The Fully Sintered, Highly Esthetic Zirconia for In-Office Use

- Recommended for anterior crowns demanding high strength and excellent esthetics
- Mills in under 35 minutes\*
- Available exclusively for Glidewell milling systems
- Optimized combination of strength and beauty



PATIENT-PLEASING  
RESULTS



NO OVEN  
NEEDED



ANTERIOR  
CROWNS



HIGH  
STRENGTH

For more information  
[glidewell.com/milling-blocks](http://glidewell.com/milling-blocks) | 844-949-7184

**glidewell.io**<sup>™</sup>  
IN-OFFICE SOLUTION

# BENEFITS

- Lifelike translucency engineered for the anterior
- Deliver restorations right from the mill with no post-processing required
- Easy delivery — conventional cementing with no need to bond
- Preserve your patients' existing dentition with minimal preparation requirements

# SPECIFICATIONS

## MATERIAL

- Pre-shaded, fully sintered zirconia
- Flexural strength: 870 MPa

## IDEAL FOR

- Single-unit anterior restorations, including bicuspid
- Cases with limited interocclusal space
- Restorations with < 1 mm margin thickness


## SHADES

- 16 VITA® Classical Shades: A1, A2, A3, A3.5, A4, B1, B2, B3, B4, C1, C2, C3, C4, D2, D3, and D4
- 2 Bleach Shades: BL1 and BL White

## READY TO DELIVER IN UNDER 45 MINUTES



# EASY PREPARATION

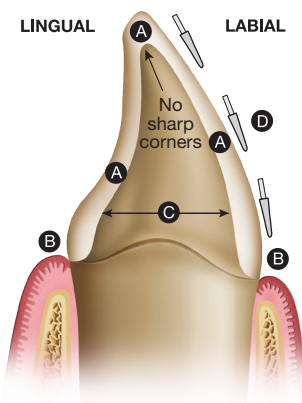

- A. 1.25 mm ideal reduction (0.7 mm minimum)
- B. Chamfer or modified shoulder margins preferred
- C. Axial walls must be convergent (avoid undercuts)
- D. Preparation should be cut in three planes
- E. To achieve optimal impression quality, gingival retraction is necessary for preparations with subgingival or equigingival margins

For more information  
[glidewell.com/milling-blocks](http://glidewell.com/milling-blocks) | 844-949-7184



*for the sake of smiles*

\*Milling times vary based on prescription and mill used. Average milling time shown reflects use of the fastmill.io™ In-Office Mill. VITA is a registered trademark of VITA Zahnfabrik.